

TrueMinerals

Essential Product to Maintain A Healthy Smile



Our TrueMinerals treatment is an amazing product, with over 73 trace minerals and other key ingredients that are essential to healthy teeth and enamel. This quick and easy treatment is highly beneficial and recommended for all clients. This treatment will replenish teeth with vital minerals to:

- Strengthen enamel
- Repair damaged enamel
- Safeguard against enamel erosion
- Help prevent tooth decay
- Eliminate tooth sensitivity
- **Reverse the effects of decalcification (see attached Tooth Decalcification Sheet)**


DESENSITIZING DURING TEETH WHITENING: Brush a thin layer of gel on the teeth and gums before and after the whitening application. Leave gel on the teeth for 10 minutes then rinse and start the whitening process. Apply again after the whitening is completed.


ENAMEL REPAIR, DECALCIFICATION, AND GENERAL TOOTH HEALTH: Brush a thin layer of gel onto the top and bottom teeth. Do not rinse. Let the gel absorb into the saliva. Recommended use, a minimum of once a week.

Davinci
Teeth Whitening System

720-596-4628, Opt 1
www.davinciteethwhitening.com

Follow us on:

 @davinciteethwhitening

 @davincicorporate